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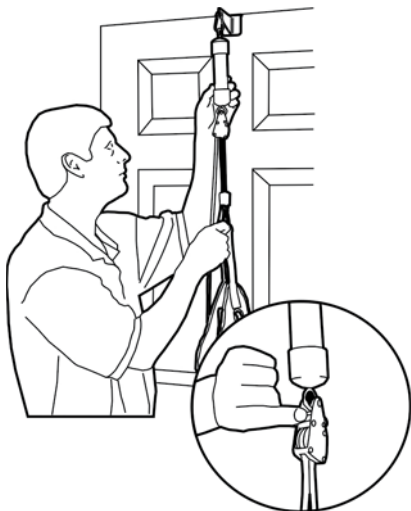
Mount metal bracket top and center of door.  
 Engage release tab on bracket and lower harness for applying to head and neck.

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Sit in chair with your back firmly against back of chair, and the chair pushed back against door.  
 Assume position of comfort and keep same position in order to maintain static tension traction.

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Apply harness to back of head and chin. For comfort, place chinstrap so that it embraces the forward structure of the chin in a cupping or concave manner.  
 Pull hook and loop strap forward and attach to patches on both sides of the chinstrap. Ensure a firm grip to back of head and neck.

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Patent # 7,144,380  
 NECKPRO is a registered trademark of  
 Gilliam Enterprises, LLC

Tighten harness by pulling down on white cord. Ease up on cord after each click to sense the increased tension/traction. When you sense the first tug or lift, begin counting the clicks. Each click equals one (1) pound of traction or tension.

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Release/remove harness. DO NOT release harness by reaching up to tab on ratchet, but rather, grasp forward end of hook and loop straps, pull back and up behind head for ease of removal.

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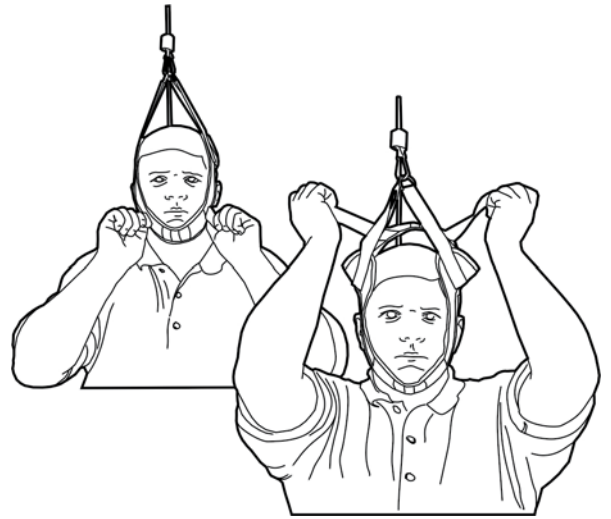
**Attention:** Use as directed by your licensed medical practitioner to be sure you have no medical or physical condition, which would preclude its use. Use only according to the above instructions. DO NOT modify or disassemble. Keep away from children.

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©2008 Gilliam Enterprises, LLC
©2008 Gilliam Enterprises, LLC • P.O. Box 5685 • Knoxville, TN 37928 USA • 1-866-655-0517 - 1-865-686-9401 FAX